

PHOVIA LIGHT THERAPY PROCESS

STEP 1:

Your vet will prepare your pet's skin to ensure optimum contact between the gel and the skin. This will usually require shaving your pet's hair.



STEP 2:

Your vet will mix the specially formulated topical gel which is orange in colour.




STEP 3:

The gel will be applied in a thin layer to your pet's skin.



STEP 4:

The PHOVIA LED light lamp will be positioned closely to the skin. After 2 minutes, the lamp will automatically turn off.

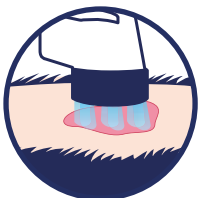

STEP 5:

The gel will be wiped off with a simple saline solution.



STEP 6:

It is likely that your vet will recommend two applications of PHOVIA in one visit. If this does not suit you, or your pet is too wriggly, it is possible to return to the practice for a second application later in that same seven-day period.



It is important to follow all of your vet's recommendations with regards to the management of your pet's skin very closely, including the regularity of your PHOVIA light therapy sessions.

CONCERNS AFTER PHOVIA:

Whilst it is very unlikely that your pet will experience any issues, there is the potential for the following complications associated with the PHOVIA system: hair colour change, reddening of the skin that regresses within 6-12 hours, skin hyperpigmentation, pain or burning at the wound site, and fatigue. Please contact your vet immediately if you have any concerns regarding your pet.

Topical gel application only to be used with Phovia LED lamp. This equipment/ system is intended for use by a veterinary professional or a suitably trained person under his/her supervision only. Animal use only.

For more details, please contact:

+44 (0) 1280 814500

uk_office@vetoquinol.com

www.vetoquinol.co.uk



Art 9400



PHOVIA LIGHT THERAPY FOR PETS



DIFFERENT COLOURS HAVE DIFFERENT EFFECTS

BLUE LIGHT

Controls bacteria and
reduces inflammation

GREEN LIGHT


Stimulates fibroblasts and
reduces inflammation

YELLOW LIGHT

Increases ATP and blood circulation

RED LIGHT

Induces dissociation of inhibitory nitric
oxide and regulates collagen production

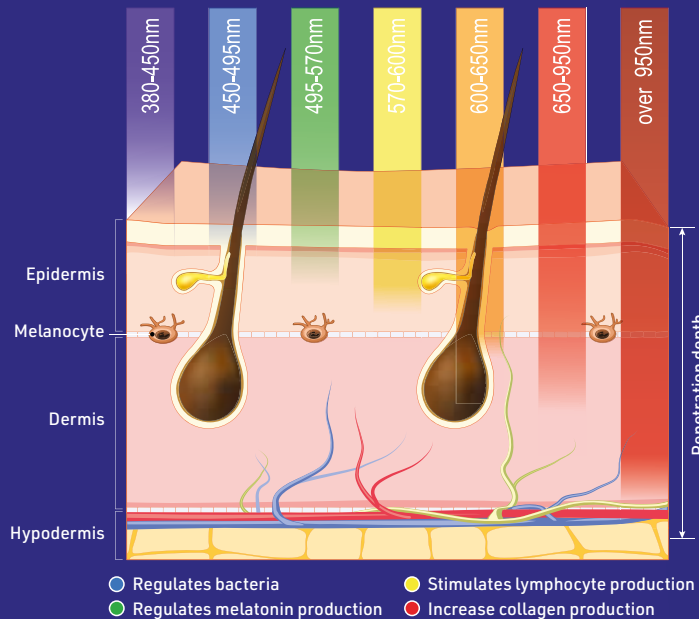

Your vet has recommended
PHOVIA® - a gentle and
non-invasive light therapy
which helps heal skin lesions
and wounds up to twice as
fast with light



WHAT IS PHOVIA LIGHT THERAPY?

PHOVIA is an innovative light therapy system which consists of a specially formulated topical gel and an LED light lamp which when used in combination, support the skin's natural regeneration process.

WHAT CAN I EXPECT FROM A PHOVIA LIGHT THERAPY SESSION?

First the topical gel is applied to the skin, and then the LED light lamp is used to illuminate the area for a period of two minutes. The gel will then be removed from the skin, and the process will be repeated for optimum results.

To ensure good contact can be achieved between the gel and the skin, it is usually necessary to shave your pet's hair in the area where the gel will be applied.



PHOVIA